

Neotech ChinStrap™

Respiratory Support Accessory

INDICATIONS FOR USE

The Neotech ChinStrap is intended to help keep a patient's mouth closed during respiratory therapy. It is intended for use on pediatric (neonates, infants and children) patients.

DIRECTIONS FOR USE

Step 1

Select proper size by measuring circumference from center of chin, over the ear, to posterior of head.

Step 2

Place Neotech ChinStrap across chin with indentation facing neck and smooth, white side against skin.

Step 3

Position wider, lower portion of split chin strap under chin and upper portion across front of chin.

Step 4

Pull strap toward posterior of head with ears positioned inside ear openings and gently fasten hook tab to strap. Side marked "LEFT" should be under patient's left ear.

Note: May be trimmed for proper fit. Ensure that strap is not too tight or pressing on ears. Hook tab should completely overlap strap.

Step 5

Replace every 5 days or per hospital protocol, whichever is sooner.

Replace immediately if worn or visibly soiled.

GENERAL INFORMATION

Product Materials

NeoFoam, hook fabric
Made with natural rubber latex: No
Made with plasticizer DEHP: No
Consult MSDS for complete material information.

Size

N784 Nano: fits head circumference 22-27 cm
N785 Micro: fits head circumference 28-35 cm
N786 Small: fits head circumference 33-42 cm
N787 Large: fits head circumference 36-46 cm

Packaging

Individually packaged: Yes
Package type: Polyethylene bag
Dispenser box: Yes

Visit neotechproducts.com for the most up-to-date information or to download the Material Safety Data Sheet (MSDS).

Polyethylene bags are biodegradable. Boxes are printed with environmentally friendly soy ink on recyclable material.

CAUTIONS & WARNINGS

The Neotech ChinStrap is not intended for use on adults.

Federal law restricts this device to sale by or on the order of a physician.

Discontinue immediately if skin irritation occurs.

Single patient use only.



For more information or to order your FREE sample call
1-800-966-0500 or visit neotechproducts.com

Neotech Products, Inc. • 28430 Witherspoon Parkway • Valencia, CA 91355