NeoBond[®]

Hydrocolloid Adhesive Strips

REF Catalog No.

DIRECTIONS FOR USE
INSTRUCTIONS D'UTILISATION

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N741 Pre-cut Strips
N742 Roll

Manufactured by
NEOTECH PRODUCTS, INC.
28430 Witherspoon Parkway, Valencia, CA 91355, USA
1-800-966-0500 • neotechproducts.com

EMERGO EUROPE
Prinsessegracht 20
2514 AP, The Hague, The Netherlands

Made in USA

D490 Rev C

English
NeoBond[®]
Hydrocolloid Adhesive Strips

FEATURES & BENEFITS

- Skin friendly alternative to acrylic tape
- Developed for long term attachment
- Ideal for use with limb boards
- Not made with natural rubber latex or plasticizer DEHP

INDICATIONS FOR USE

NeoBond Hydrocolloid Adhesive Strips are intended for general use as a medical adhesive.

DIRECTIONS FOR USE

Preparation

Clean and dry skin per hospital protocol.

Note: It is important for skin to be as clean and dry as possible. NeoBond will not adhere properly to moist skin or hair. Oils and lotions will also adversely affect adhesion.

DO NOT USE ALCOHOL.

Application

Step 1 Before removing liner, warm NeoBond in hands for 60 seconds. May also be held under radiant or heel warmer for 10 to 15 seconds.

Step 2 Peel and discard liner. Wrap NeoBond Strip around skin and product and hold in place for 60 seconds to ensure proper adhesion.

Removal

Step 1 Replace NeoBond Strip every 5 days or per hospital protocol, whichever is sooner.

Step 2 Saturate NeoBond Strip with water or saline.

Step 3 Slowly peel NeoBond Strip away from skin as you swab with water or saline.

CAUTIONS & WARNINGS

Discontinue immediately if skin irritation occurs.

NeoBond will not adhere properly to moist skin or hair. Oils and lotions will also adversely affect adhesion.

Do not use alcohol to clean skin prior to application.

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