


# EZ-Hold<sup>®</sup>


## Tubing and Cannula Holder

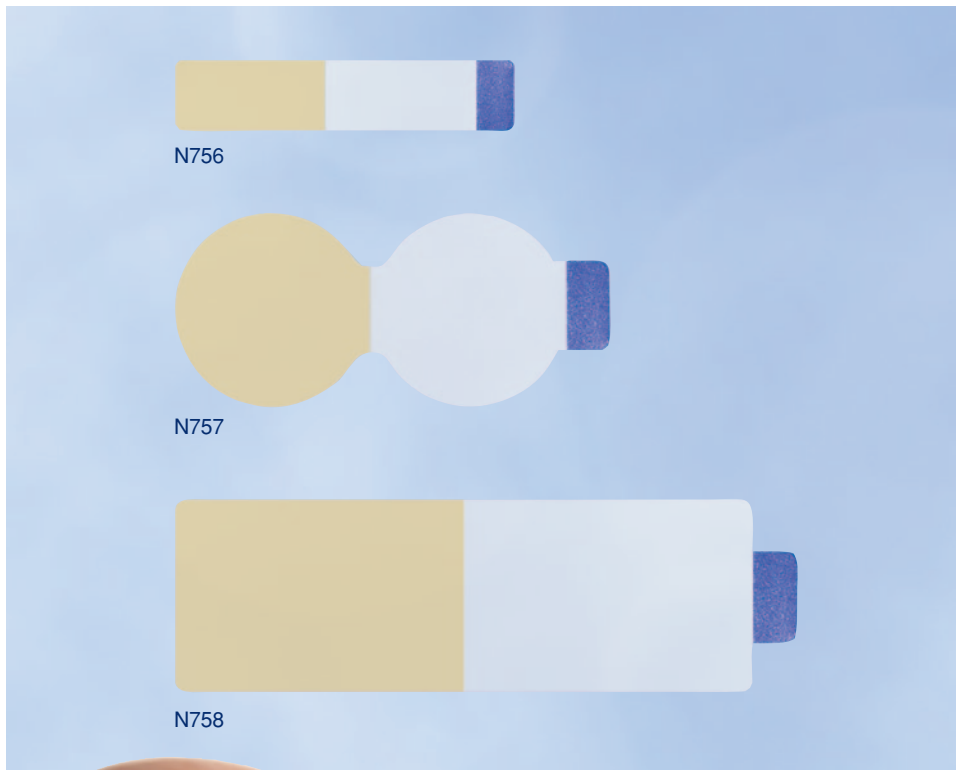

### FEATURES AND BENEFITS

- Skin friendly NeoBond<sup>®</sup> Hydrocolloid base
- Fold over flap engineered to relieve stress on tubes
- Flap designed to be opened and closed repeatedly
- Clear flap allows for better visibility
- Mini size and shape ideal for micro preemies
- Not made with natural rubber latex or plasticizer DEHP

Cat No.	Item	Qty/Unit
N756	Mini	100/box
N757	Standard	100/box
N758	Extra Large	100/box

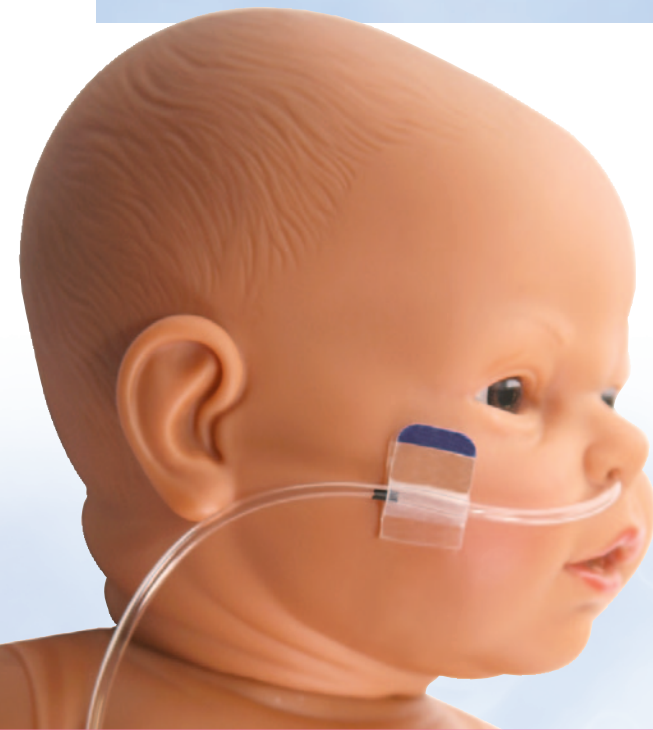

Made in USA

U.S. Patent #D506,547

# EZ-Hold®

## Tubing and Cannula Holder

### INDICATIONS FOR USE

---

The EZ-Hold is intended to anchor and support breathing tubes.

### DIRECTIONS FOR USE

---

#### Preparation

Clean and dry skin per hospital protocol.

Note: It is important for skin to be as clean and dry as possible. EZ-Hold will not adhere properly to moist skin or hair. Oils and lotions will also adversely affect adhesion.

DO NOT USE ALCOHOL.

#### Application

##### Step 1

Before removing liners, warm EZ-Hold in hands for 60 seconds. May also be held under radiant or heel warmer for 10 to 15 seconds.

##### Step 2

Discard white, paper liner, then peel and discard clear, plastic liner from hydrocolloid base.

##### Step 3

Apply base to skin and hold in place for 60 seconds to ensure proper adhesion.

##### Step 4

Peel and discard second liner and fold flap over tubing.

Note: Lift flap as needed to reposition or remove tubing.

### Removal

##### Step 1

Replace EZ-Hold every 5 days or per hospital protocol, whichever is sooner.

##### Step 2

Lift flap to remove tubing.

##### Step 3

Saturate EZ-Hold with water or saline.

##### Step 4

Slowly peel EZ-Hold away from skin as you swab with water or saline.

### GENERAL INFORMATION

---

#### Product Materials

NeoBond Hydrocolloid, PET film, acrylic pressure sensitive adhesive (does not contact skin), rubber pressure sensitive adhesive (does not contact skin), Made with natural rubber latex: No  
Made with plasticizer DEHP: No  
Consult MSDS for complete material information.

#### Size

N756 Mini closed: 13 mm x 25 mm  
N757 Standard closed diameter: 25 mm  
N758 XL closed: 25 mm x 33 mm

#### Packaging

Standard box, no dispense feature

*Visit [neotechproducts.com](http://neotechproducts.com) for the most up-to-date information.*

*Boxes are printed with environmentally friendly soy ink on recyclable material.*

### CAUTIONS & WARNINGS

---

Discontinue immediately if skin irritation occurs.

EZ-Hold will not adhere properly to moist skin or hair. Oils and lotions will also adversely affect adhesion.

Do not use alcohol to clean skin prior to application.

Single patient use only.



For more information or to order your FREE sample call  
**1-800-966-0500** or visit [neotechproducts.com](http://neotechproducts.com)

Neotech Products, Inc. • 28430 Witherspoon Pkwy • Valencia, CA 91355, USA