

**NeoBond**<sup>®</sup>  
 Hydrocolloid Adhesive



**FEATURES AND BENEFITS**

- Skin friendly alternative to acrylic tape
- Developed for long term attachment
- N741 and N742 ideal for use with limb boards
- N7426 pre-cut heart shape design
- Not made with natural rubber latex or plasticizer DEHP

Cat No.	Size	Qty/Unit
N741	1/2" x 6" Strips	100/box*
*50 sets of 2		
N742	1" x 15' Roll	1/box
N7426 Shapes	1 1/4" x 1 1/2" Hearts	100/box



# NeoBond<sup>®</sup>

## Hydrocolloid Adhesive

### INDICATIONS FOR USE

NeoBond Hydrocolloid Adhesive is intended for general use as a medical adhesive.

### DIRECTIONS FOR USE

#### Preparation

Clean and dry skin per hospital protocol.

Note: It is important for skin to be as clean and dry as possible. NeoBond will not adhere properly to moist skin or hair. Oils and lotions will also adversely affect adhesion.

DO NOT USE ALCOHOL.

#### Application

##### Step 1

Before removing liner, warm NeoBond in hands for 60 seconds. May also be held under radiant or heel warmer for 10 to 15 seconds.

##### Step 2

Peel and discard liner. Wrap or apply NeoBond over skin and product (e.g. feeding tube, limb board) and hold in place for 60 seconds to ensure proper adhesion.

#### Removal

##### Step 1

Replace NeoBond every 5 days or per hospital protocol, whichever is sooner.

##### Step 2

Saturate NeoBond with water or saline.

##### Step 3

Slowly peel NeoBond away from skin as you swab with water or saline.

### GENERAL INFORMATION

#### Product Materials

NeoBond Hydrocolloid, polyester silicone coating, polystyrene backing

Made with natural rubber latex: No  
Made with plasticizer DEHP: No

#### Size

N741 pre-cut strip: 1/2 in. x 6 in.  
N742 roll: 1 in. x 15 ft.  
N7426 heart: 1.25 in. x 1.1562 in.

#### Packaging

N741 Strips: Strips of two on plastic liner  
N742 Roll: Continuous roll in dispenser box with coated paper liner  
N7426 Shapes (Hearts): Set of five on perforated plastic liner in polyethylene bag

*Visit [neotechproducts.com](http://neotechproducts.com) for the most up-to-date information or to download.*

*Boxes are printed with environmentally friendly soy ink on recyclable material.*

### CAUTIONS & WARNINGS

Discontinue immediately if skin irritation occurs.

NeoBond will not adhere properly to moist skin or hair. Oils and lotions will also adversely affect adhesion.

Do not use alcohol to clean skin prior to application.

Single patient use only.



For more information or to order your FREE sample call  
**1-800-966-0500** or visit [neotechproducts.com](http://neotechproducts.com)

Neotech Products • 28430 Witherspoon Pkwy • Valencia, CA 91355, USA