


NeoPulseTM

Disposable Pulse Oximeter Wrap

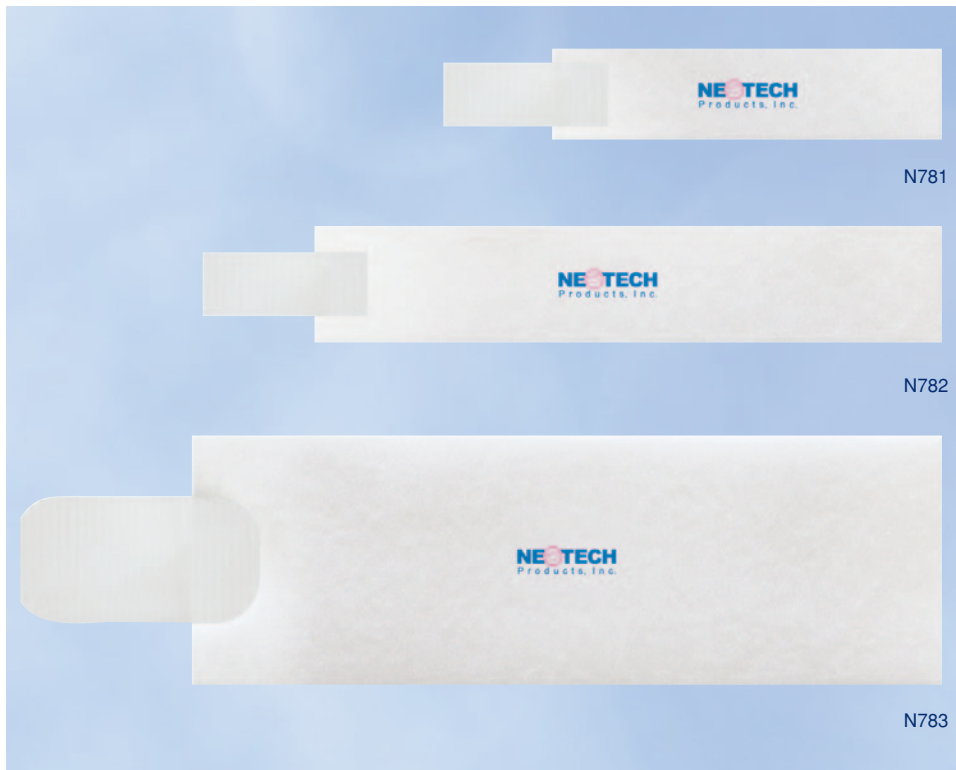

FEATURES AND BENEFITS

- Soft, breathable, skin friendly NeoFoam® wicks perspiration
- Lightweight, form fitting and adjustable
- Designed for comfort and stability
- Not made with natural rubber latex or plasticizer DEHP
- Individually packaged

Cat No.	Size	Dimensions	Qty/Unit
N781	Small	3/4" x 3"	100/box
N782	Large	1" x 5"	100/box
N783	X-Large	2" x 6"	100/box



Made in USA

NeoPulse™

Disposable Pulse Oximeter Wrap

INDICATIONS FOR USE

The NeoPulse is intended to hold a pulse oximeter sensor in place. It is intended for use on pediatric (neonates, infants and children) patients.

DIRECTIONS FOR USE

Step 1

Place pulse oximeter sensor per hospital protocol.

Step 2

Wrap NeoPulse around foot and pulse oximeter sensor and gently fasten hook tab to wrap.

Note: May be trimmed for proper fit. Ensure that wrap is not too tight. Hook tab should completely overlap wrap.

Step 3

Check foot after wrapping to ensure adequate circulation.

Step 4

Change the NeoPulse every 5 days or per hospital protocol, whichever is sooner.

Replace immediately if visibly worn or soiled.

GENERAL INFORMATION

Product Materials

NeoFoam, hook fabric
Made with natural rubber latex: No
Made with plasticizer DEHP: No
Consult MSDS for complete material information.

Size

N781 Small: 75 mm x 20 mm
N782 Large: 125 mm x 25 mm
N783 XL: 150 mm x 50 mm

Packaging

Individually packaged: Yes
Package type: Polyethylene bag
Dispenser box: Yes

Visit neotechproducts.com for the most up-to-date information or to download the Material Safety Data Sheet (MSDS).

Polyethylene bags are biodegradable. Boxes are printed with environmentally friendly soy ink on recyclable material.

CAUTIONS & WARNINGS

Discontinue immediately if skin irritation occurs.

Single patient use only.



For more information or to order your FREE sample call
1-800-966-0500 or visit **neotechproducts.com**

Neotech Products, Inc. • 28430 Witherspoon Pkwy, Valencia, CA 91355