
NeoGlo®

Transilluminator

REF Catalog No.

N555

DIRECTIONS FOR USE
INSTRUCTIONS D’UTILISATION
ISTRUZIONI PER L’USO
INSTRUCCIONES DE USO
INSTRUÇÕES DE UTILIZAÇÃO
GEBRAUCHSANWEISUNG
NÁVOD K POUŽITÍ
ZALECENIA DOTYCZĄCE
UŻYTKOWANIA
إرشادات الاستخدام

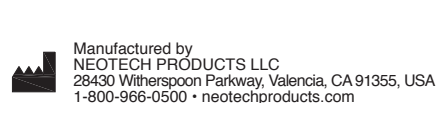Manufactured by
NEOTECH PRODUCTS LLC
 28430 Witherspoon Parkway, Valencia, CA 91355, USA
 1-800-966-0500 • neotechproducts.com

EC REP **EMERGO EUROPE**
 Prinsessegracht 20
 2514 AP, The Hague, The Netherlands

D504 Rev C

English
NeoGlo®
Transilluminator

INDICATIONS FOR USE

The NeoGlo Transilluminator is a hand-held, battery-powered device that is a light source intended to transmit light through tissues to aid examination of patients.

DIRECTIONS FOR USE

Step 1 Remove battery cover, insert AA alkaline battery (included) and replace cover.

Note: Light settings will cycle through once and return to off setting.

Caution: Do not stare directly into lights. Inserting battery in the wrong direction may produce heat and damage the unit.

Step 2 Select desired light setting:

- Press power button  once for forward facing white lights.

- Press again for upward facing white lights.

- Press again for upward facing red lights.

- Press again for upward facing white and red lights.

- Press again to turn off.

- Repeat cycle as needed.

Step 3 With preferred light setting on, hold NeoGlo to patient’s skin for illumination.

Note: Dim room lights for most effective illumination.

Step 4 Clean NeoGlo with alcohol wipe or CaviWipe™ after each use.

CAUTIONS & WARNINGS

Do not shine light directly into eyes.

Do not submerge NeoGlo in liquid.

Do not autoclave NeoGlo. Exposure to steam or excessive humidity may cause electrical failure.

Use AA alkaline batteries only. Do not insert rechargeable batteries. Inserting battery incorrectly may damage the unit and void the warranty.

Dispose of NeoGlo in accordance with local electronic regulations.

30 DAY SATISFACTION GUARANTEE

If you are not satisfied with the performance of your NeoGlo for any reason, simply return within 30 days of purchase. Send back your NeoGlo in its original packaging along with proof of purchase and Neotech will refund the full purchase price, less shipping. Shipping cost will only be refunded if the product is defective.

Customer is responsible for return shipping. We do not accept returns for any reason after 30 days. If your unit is not working properly after 30 days, please refer to WARRANTY.

NeoGlo Transilluminators purchased through a third party seller are subject to their return policy.

WARRANTY

One year warranty on electronic performance based on normal intended use and storage (-40°C/-40°F to +85°C/+185°F; relative humidity between 5% and 95%, non-condensing).

Exclusions; warranty does not cover:

a) Battery replacement

b) Damage or failure caused by leaking batteries, rust, corrosion, water and other liquid damage, infestation, negligence, abuse, or misuse

c) Failure to follow manufacturers recommended operating and maintenance procedures

NeoGlo must be returned in its original packaging along with proof of purchase. Failure to return without the original packaging may void the warranty.

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